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
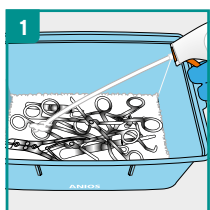
Ready-to-use pre-treatment disinfectant detergent foam for optimised dissolving of contamination on instruments with longer standing times (up to 72 hours) or before machine processing.

Prevents drying out of contamination and makes pre-cleaning easier with a good cleaning effect. The thick foam guarantees even coverage of the instrument surface. Aniosyme Prime is compatible with a wide variety of materials. Not suitable for instruments made of zinc or iron.


KASUTUSJUHEND

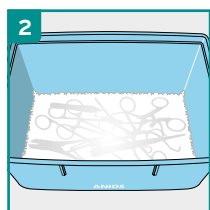
1 Beforehand, spray the bottom of the soaking tray with Aniosyme Prime and place instruments collected.

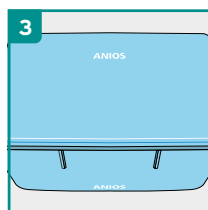2 Re-cover all the instruments with foam. Minimum contact time in the system: 15 minutes. For tuberculocidal activity, allow 30 minutes contact time.

3 Cover the tray with the lid.

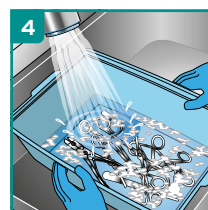4 Thoroughly rinse treated medical devices with tap water to remove all traces of product and organic soiling before machine or manual cleaning.

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Distributed by:
LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

ECOLAB

LABORATOIRES ANIOS
An Ecolab Company

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Pre-treatment of surgical instrumentation in foam: during a dry mode of transport or during storage of soiled instruments before machine or manual washing. Maintains an adequate level of humidity to avoid blood and other organic materials drying.

KASUTUSJUHEND

Ready-to-use solution.

1. Beforehand, spray the bottom of the soaking tray with Aniosyme Prime and place instruments collected.
2. Re-cover all the instruments with foam. Minimum contact time in the system: 15 minutes. For tuberculocidal activity, allow 30 minutes contact time.
3. Cover the tray with the lid.
4. Thoroughly rinse treated medical devices with tap water to remove all traces of product and organic soiling before machine or manual cleaning.

Not suitable for instruments made of zinc or iron.

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SÄILITAMINE

Dangerous - respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products). Storage: between +5°C and +25°C.

SERTIFIKAADID

Medical device. Full antimicrobial activity for Aniosyme Prime is included in the scientific dossier, available on request.

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